Trilly

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DMT S.r.l. can provide dentists with the widest not only commercial, but also technical and clinical -consultance on lasers, for solving all the problems linked to the use of laser in dentistry.

Research and development are the constant aims in order to create new solutions. Trilly whitening system comes from the need to provide a suitable stable whitening product whitening laser assisted.

Trilly also thanks to its versatility, it is also suitable for use with the most common lamps for bleaching. Trilly stands for quality and effectiveness and cost-effectiveness at the highest levels of quality and functionality. Trilly has been designed and produced absolutely in accordance with the present rule, requirements for medical devices.

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Effectiveness

Trilly is a hydrogen peroxide enriched to 38% of red dyes and molecular components that activate special fast exchange oxygen ions during the laser irradiation.

The peculiarity of such molecular components that make Trilly is active immediately 1 minute after the laser exposure.

Fast

Thanks to their molecular composition, Trilly is particularly fast, is possible, with assisted laser treatment, make only with one application of product and 40 seconds of irradiation for a tooth (10 min. entire mouth), you can reach the optimum level of whitening. (From 3 to 9 scale tones Vita).

Versatility

The chemistry of Trilly it still allows a use to non- laser users. Trilly is also functional with common bleaching lamps on the market are achieved excellent results with 2 applications of product. Not last Trilly can also be used without any light source leaving it for 30/40 minutes on teeth.

Affordability

The high effectiveness of Trilly also allows a great saving to the user. In fact, with 6 syringes of product packaging can be treated with great effectiveness at least 12 patients (min 12 - max 18).

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
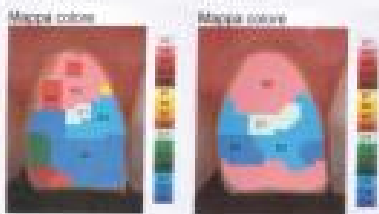
Clinical Case Report 1

The patient shows after the color analysis, a clear preponderance of color D3, after a single application of Trilly goes to the entire arch in C1 tone, but obtains into the incisal zone a surprising A2 combined at the original of the cervical area.

Even an in-depth analysis of the color mapping point to point highlights as well as an obvious improvement in chroma, you have obtained a uniform color.

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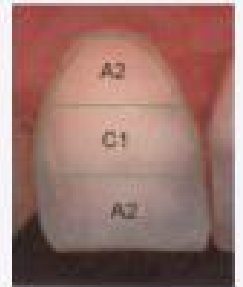
Clinical Case Report 2

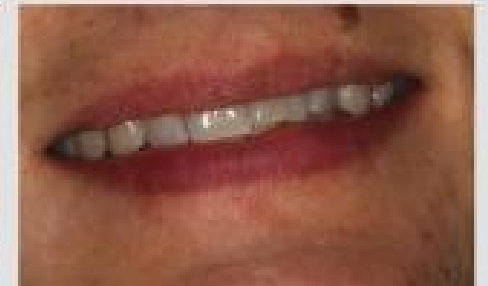Full arch bleaching, by the method of individual transparent mask. The patient starts from a Chroma B3 and reaches a chroma B1, with a single application of the product, even in this case with laser therapy assisted.

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After having protected the soft tissues with liquid dam protection, apply the product evenly on the tooth, normally Half syringe is sufficient for 2 arches from canine to canine.

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With diode laser in continuous wave with 2 watts of power irradiate for 40 seconds every tooth. After lasering leave on for three minutes.

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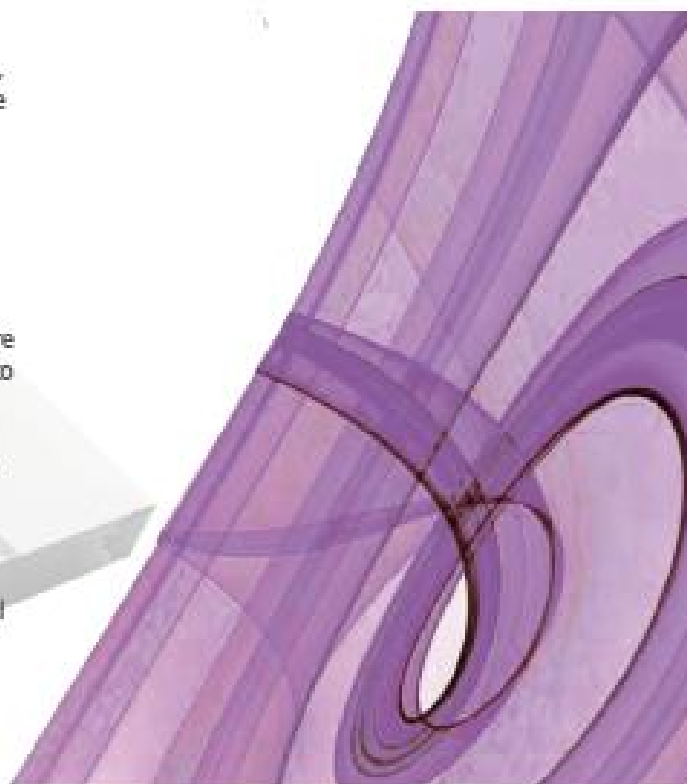
Apply the product and follow the cycles of irradiation suggested by producer.

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Apply the gel and remove after 30 minutes

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